

User Guide VaPro Plus Pocket™ Catheters



2

1. Assemble all equipment and prepare the area.
2. Wash your hands as advised by your healthcare practitioner.
3. Position yourself to carry out intermittent catheterisation. Manipulate your clothing to access your genitalia.




4

4. Open the package. To open package, pull top finger hole toward you and all the way down.



6

5. Remove catheter from package.
6. With collection bag facing up, hold the catheter in one hand while pushing the bag away from the catheter with the other hand to break the paper band. Unfold collection bag. Leave the catheter on the lap, sink, or other flat surface.

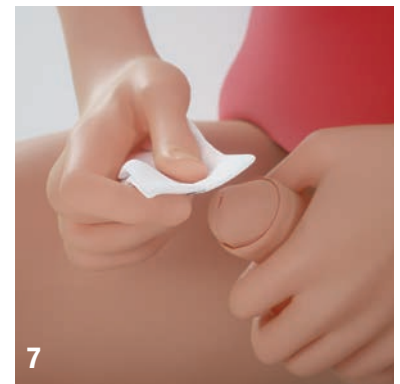


7

7. Hold the penis with your non-dominant hand and don't let go until you have finished the procedure. Pull back foreskin (if present) and cleanse the glans and opening of urethra as instructed by your healthcare practitioner.



8

8. Remove the cap from the protective tip of catheter.

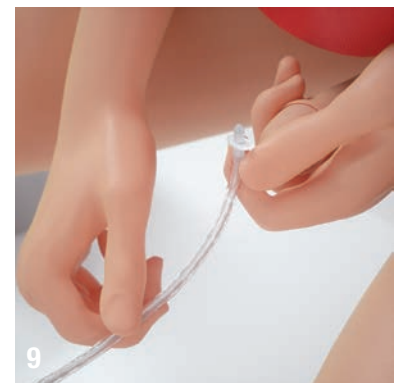

9

9. Hold the catheter in one hand, and with the other hand advance the catheter forward until tip of the catheter fills the protective tip, taking care that the catheter does not protrude from the tip.


10. Grasp catheter below protective tip plate.

Featuring HydraBalance™
Lubricating Technology




HydraBalance™
Lubricating Technology

VaPro Plus Pocket™
No Touch Intermittent Catheter

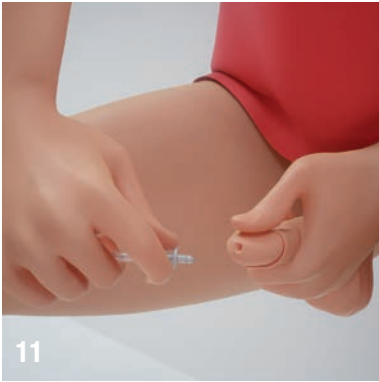

11. Using your forefinger and thumb of your non-dominant hand squeeze penis gently to open the urethral opening. With your dominant hand, insert the protective tip until the plate comes in contact with urethral opening.

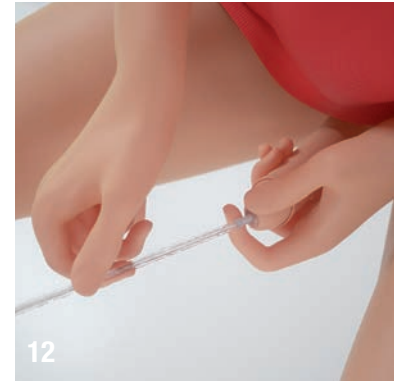

12. With the penis stretched and in upright position, insert the catheter forward until urine starts to flow.



13. Once urine flow has stopped, slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely. Reposition the foreskin (if present).

14. After catheterising hold collection bag by handle.

15. To empty the collection bag before disposing, tear at the arrow (Tear here to empty) to drain.



16. Catheter and integrated collection bag may be disposed of in a waste bin. DO NOT FLUSH DOWN TOILET.

17. Wash your hands.

Not made with natural rubber latex. Intermittent catheter not made with PVC or phthalates. This product consists of: polyurethane sleeve, polyethylene and elastomeric polymer collection bag, and thermoplastic elastomer tip and tubing.

Indications and Intended Use: This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and paediatric patients who need to drain urine from the bladder.

WARNINGS Do not use if package is damaged or opened. To help reduce the potential for infection and/or other complications, do not reuse. If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

Precautions: Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Pronounced stricture
- Urethritis – inflammation of the urethra
- Epididymitis – inflammation of the epididymis (testicle tube)
- Unexplained urethral bleeding
- False passage
- Prostatitis – inflammation of the prostate gland

Self-catheterisation should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterisation frequency varies by person, the recommended frequency of your catheterisation should be provided by your healthcare practitioner. For any other questions about your catheterisation, please contact your healthcare practitioner.

NOTE: Store boxes at normal room temperature. Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact your local distributor.

Rx Only

Hollister Australia

PO Box 375,
Box Hill, VIC 3128
1800 880 851
priority@hollister.com.au
www.hollister.com.au

Hollister New Zealand

PO Box 107097,
Auckland Airport, Auckland 2150
0800 678 669
NZcustomerservice@libmed.co.nz
www.hollister.co.nz

AUS



NZ



Scan to watch video