
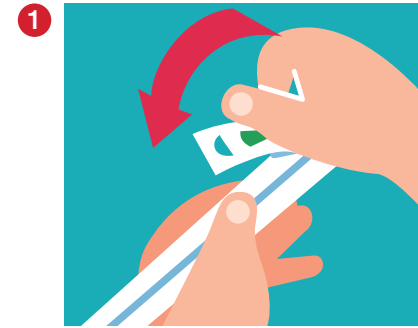
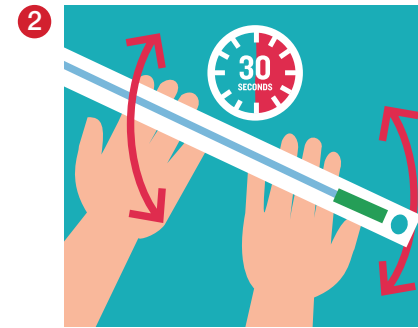
Boys' Guide for Use

1 Release the water.
Keep the package closed, and fold the water pack in half. Squeeze the pack with your fingers or press it down on a firm surface. The pressure will cause the pack to burst and release the water.

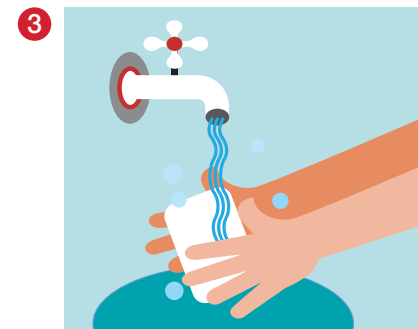2 Wet the catheter.
Hold the catheter package and tip it up and down six times, so the catheter will get wet inside. Wait at least 30 seconds from when you wet the catheter before using it.


3 Wash hands with soap and water.


4 Open the package.
Using the holes on the catheter package, peel the package open about half way.

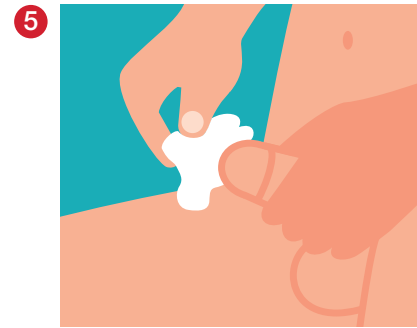Hint: The sticky strip on the catheter package can be used to stick the package to a surface to hold the catheter while you get ready for the next steps.


5 Hold your penis, and pull back the foreskin (if you have one). Clean the tip of your penis with soap and water, or a cleansing wipe.

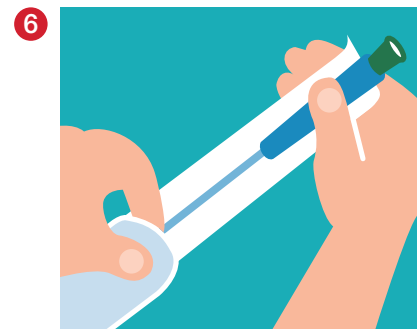Hint: Don't squeeze your penis too hard. This can block the opening and make it difficult to insert the catheter.


6 Remove the catheter from the package. Hold on to the funnel and/or the sliding sleeve.

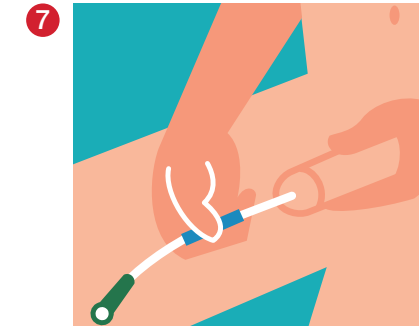Hint: The catheter is very slippery! Use the sliding sleeve to help you hold onto the catheter and to help you not touch the catheter with your fingers.


7 With one hand, hold your penis facing up. Insert the catheter into the tip of your penis until pee starts to flow. Be sure the open end of your catheter is directed into the toilet.

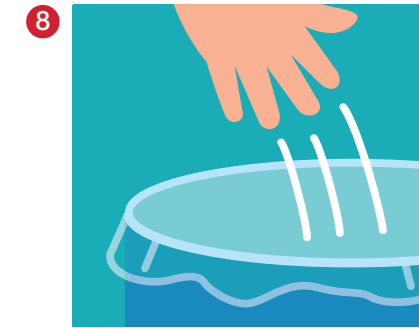When the pee stops, slowly pull the catheter out. Pee may start to flow a little bit more. That is normal. Just pause and wait for the pee to stop again before removing the catheter.

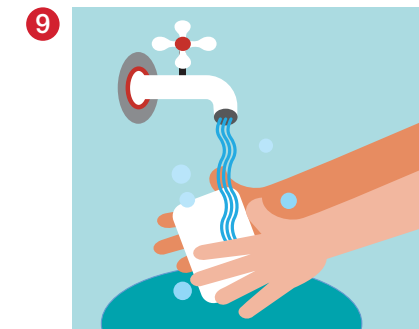8 Throw the catheter away in the garbage. Do not flush it down the toilet.

9 Wash your hands again with soap and water.

- a** Hydrophilic Coating
- b** Sliding Sleeve
- c** Burstable sterile water pack

This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female, and pediatric patients who need to drain urine from the bladder.

WARNING: To help reduce the potential for infection and/or other complications, do not reuse. If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

PRECAUTION: Please consult a medical professional before using this product if any of the following conditions are present: Severed urethra, unexplained urethral bleeding, pronounced stricture, false passage, urethritis—inflammation of the urethra, prostatitis—inflammation of the prostate gland, epididymitis—inflammation of the epididymis (testicle tube).

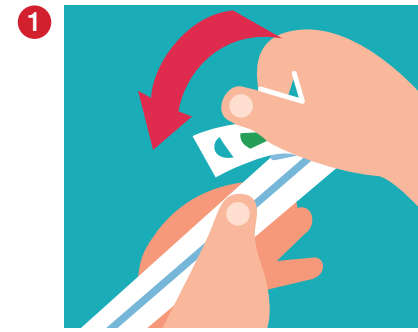
Self-catheterization should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterization frequency varies by person, the recommended frequency of your catheterization should be provided by your healthcare practitioner. For any other questions about your catheterization, please contact your healthcare practitioner.

CAUTION: RX ONLY. Federal (USA) Law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product. Prior to use of this device, be sure to read (i) the complete information on how to use this device including Warnings, Cautions, and Instructions for Use, and (ii) all other package inserts and labels supplied with the product.

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