

# Adapt Medical Adhesive Remover Spray/Adapt Universal Remover Wipe

## General Information

- Helps remove adhesive skin barriers and/or tape
- Removes adhesive residue from the skin
- Skin Friendly Formulation
  - Silicone-based
  - Alcohol free
  - No sting



**Adapt Medical Adhesive Remover Spray/Adapt Universal Remover Wipe**  
Stock No 7737/7760

## Using Adhesive Remover Spray

1. Peel back one edge of the skin barrier or tape border.
2. Hold the can so that the arrow faces the base of the skin barrier or tape, and spray until the adhesive releases. The 360° spray technology minimizes waste, and will allow you to spray from any angle.
3. Ensure your skin is dry and apply your new ostomy pouching system as usual.



**Spray** - Spray until the skin barrier releases.

## Using Universal Remover Wipes

1. Tear open the package and take out the wipe.
2. Press or rub the wipe against your skin as you peel off your skin barrier and/or tape.
3. Press or rub the wipe over any areas of the skin where tape residue remains.
4. Ensure your skin is dry and apply your new ostomy pouching system as usual.



**Wipes** - Rub the wipe over skin as you peel off skin barrier.

FREQUENTLY ASKED QUESTIONS	
Is an adhesive remover always needed?	No, this product is optional. Use it to remove adhesive residue from your skin, or to make it easier to remove your skin barrier or tape.
Will this be uncomfortable to use?	This silicone-based product is formulated to be a no-sting alternative. When used correctly, the adhesive remover will help to make the removal of your pouching system easier and help reduce the potential for skin stripping.
Do I need to wash the adhesive remover off of my skin?	The adhesive remover evaporates quickly and dries without residue, so the next barrier can attach securely. If you wash your skin after use be sure to rinse and dry thoroughly before the next pouch application.



**Wipes** - Clean off remaining tape residue.

# Adapt Medical Adhesive Remover Spray/Adapt Universal Remover Wipe

## Additional Notes:

---



---



---



---



---



---



---



---



---



---



---



---



---



---



---



---

Routine follow-up with your healthcare professional is recommended.

**Indications and Intended Use:** This product is an accessory to an ostomy pouching system intended to ease removal of an ostomy skin barrier or adhesive residue.

**Precautions:** For external use only. Use only as directed. Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, and vagina). Discontinue use if irritation occurs. Pressurised (pressurized) container. Protect can from sunlight and do not expose to or store at temperatures exceeding 122°F (50°C). Do not puncture or incinerate container, even after use. **DO NOT USE IN CONFINED SPACES.**

**Instructions for Use:**

1. Spray remover around the edge of the adhesive.
2. After a few seconds gently peel the adhesive away from the skin using additional medical adhesive remover if necessary. Dries in seconds. Once dried, medical adhesive remover will not affect adhesion of products applied later.

**Alternate Application:** Saturate a gauze pad and wipe the adhesive away. Remover evaporates quickly, additional applications to gauze may be required. May use until can is empty. Please follow all local disposal regulations.

**EXTREMELY FLAMMABLE:** Do not spray on or near a flame or any incandescent material, electrical appliance or source of heat. Keep away from sources of ignition – No Smoking. Keep out of the reach of children.

**Precaution:** Single patient – multiple use. Sharing the product with multiple users creates risk of infection.

**Notes:** Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see [www.hollister.com/authority](http://www.hollister.com/authority) or contact EC Rep or local distributor.

**Hollister Incorporated**  
 2000 Hollister Drive  
 Libertyville, Illinois 60048  
 1.800.323.4060

Hollister, the Hollister logo, and Adapt are trademarks of Hollister Incorporated.  
© 2022 Hollister Incorporated. 923073-0322 US-01716

[www.hollister.com](http://www.hollister.com)

