VaPro Pocket™

en No Touch Urethral Intermittent Catheter

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en INSTRUCTIONS FOR USE

Contents
No touch urethral intermittent catheter
Product Description

- Ready-to-use hydrophilic intermittent catheter with colour (color)-coded funnel.
- Protective tip and sleeve supports no touch insertion technique.

Catheter effective length: the length of the catheter that can be inserted into the urethra.			
20 cm / 8" catheter	17 cm	170 mm	6.7"

Not made with natural rubber latex. This product consists of: polyurethane sleeve, thermoplastic elastomer tip, and tubing not made with DEHP.
Indications and Intended Use: This intermittent catheter is a flexible tubular device that is inserted through the urethra by female and female paediatric (pediatric) patients who need to drain urine from the bladder.

WARNINGS
Do not use if package is damaged or opened. To help reduce the potential for infection and/or other complications, do not reuse. If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

Precautions: Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Unexplained urethral bleeding
- Pronounced stricture
- False passage
- Urethritis – inflammation of the urethra

Self-catheterisation (catheterization) should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterisation (catheterization) frequency varies by person, the recommended frequency of your catheterisation (catheterization) should be provided by your healthcare practitioner. For any other questions about your catheterisation (catheterization), please contact your healthcare practitioner.

NOTE: Store boxes in a flat position and at normal room temperature. Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

Urethral Intermittent catheter

Single sterile barrier system

Consult Instructions for Use if package is damaged

CE mark as required

Rx Only

Store box with arrows facing up

Number of units

Medical Device (as applicable)

Caution: Federal (USA) Law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product.

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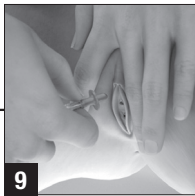
For reference only. Unauthorized use, disclosure or copying is prohibited.

en Instructions

To prepare

1. Assemble all equipment and prepare the area.
2. Wash your hands as advised by your healthcare practitioner.
3. Position yourself to carry out intermittent catheterisation (catheterization). Manipulate your clothing to access your genitalia.
4. To open package, pull toward you at the notch. Remove the catheter from the package. Leave pack on the lap, sink or other flat surface. Lay catheter on opened package.

5. Spread the labia with two fingers of your non-dominant hand. Cleanse around opening of urethra as instructed by your healthcare practitioner.

6. Remove the cap from the protective tip of catheter.
7. Hold the catheter in one hand, and with the other hand advance the catheter forward until tip of the catheter fills the protective tip, taking care that the catheter does not protrude from the tip.
8. Grasp catheter below protective tip plate.

9. Part the labia with two fingers and insert the protective tip until the plate comes in contact with urethral opening.
10. Relax the labia and use the fingers of your non-dominant hand to hold the protective tip in place as you insert the catheter with your dominant hand until the urine starts to flow.
11. Once urine flow has stopped, slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely.

To dispose

12. Catheter may be disposed of in a waste bin. DO NOT FLUSH DOWN TOILET.
13. Wash your hands.

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©2022 Hollister Incorporated.